



GINGERLEAD[®]



Dog Support & Rehabilitation Harness

Instructions

1. Pass the sling under your pet's belly holding the straps on each side of your dog. The handle should always be on the left side of your pet.
2. Hold the handle with attached leash on top. The corduroy lining in the handle should be facing down. Align the free strap on the right side of your dog under the handle.
3. Wrap the handle under and around the free strap and close the ends of the handle together so that the free strap is inside the handle and the leash is on top.
4. Attach the snap hook from the leash to your dog's collar or a standard chest harness. Now you're ready to go!



ADJUSTABLE The GingerLead is adjustable to fit various heights of dogs and owners.

Support Pad

Sliders, attached to the straps on each side of the support pad, allow the straps to be lengthened or shortened. Position the sliders at the same location on each side of the support pad to keep the strap length the same on both sides. The handle moves along one strap. The slider feeds through the handle when needed. Once adjusted, slide the handle to the top.

Leash

Shorten or lengthen the leash through the plastic buckle that attaches to the handle. A strap clip slides along the leash to hold the loose end.

IMPORTANT: Always follow instructions from your veterinarian for your dog's specific condition. Never leave on an unsupervised dog. Not intended to be left on when not in use.

Washing Instructions

Prepare Open the support pad and lay it flat. Disconnect the leash strap using the side squeeze buckle and close the handle.

Wash The GingerLead can be hand or machine washed, and air or machine dried. We do not recommend the use of bleach or harsh detergents.

Dry Tumble dry low or air dry by hanging the GingerLead from one end or laying the GingerLead flat. Machine drying may cause some shrinkage.