


Dr. Buzby's ToeGrips®

Patent Pending

Measuring the nails is an essential, one-time process for each patient. Improperly sized, ToeGrips may quickly fall off or migrate into the nail bed. For best results, nails should be appropriately trimmed before sizing and applying ToeGrips.

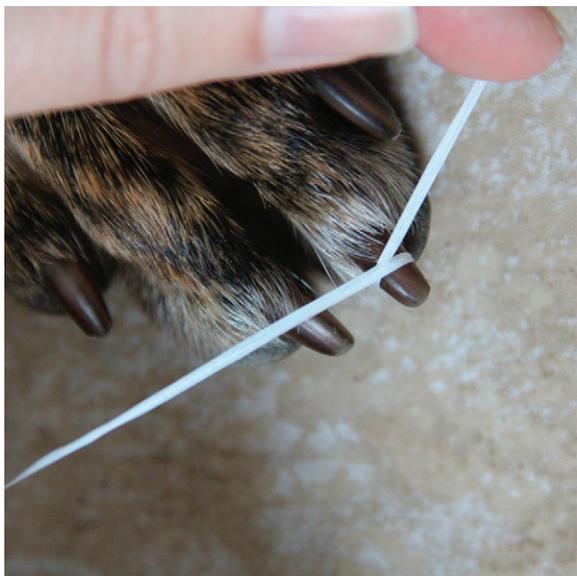

Step 1: Wrap a piece of dental floss around the midshaft of the toenail.


Step 2: Hold the floss, wrapped around the nail circumference, with one hand.

Step 3: Use a Sharpie® to mark across both pieces of floss. This will generate two points.

Step 4: Stretch the floss and measure the distance between the two points in millimeters.*

Step 5: Repeat for all weight-bearing nails on one front and one back paw.


Step 6: Based on the range of measurements obtained (not the average), use the chart* to select the proper size. Occasionally, a dog may require a combination of two sizes.

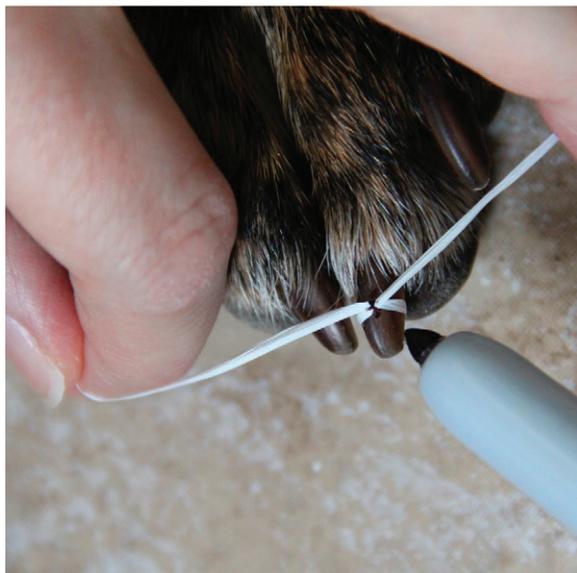

Nail Circumference	Size	Color
9-12 mm	XS	Yellow 🐾
13-15 mm	S	Red 🐾
16-19 mm	M	Blue 🐾
20-23 mm	L	Green 🐾
24-27 mm	XL	Purple 🐾
28-30 mm	XXL	Orange 🐾
31-36 mm	XXXL	Navy 🐾

Application instructions are located inside each package.

*A millimeter ruler and size chart are on the back of each package.

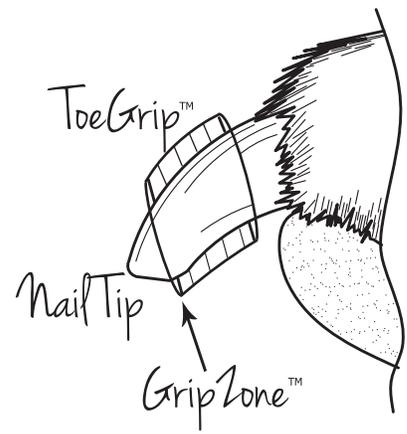

IMPORTANT

Patient Selection

Designed for senior, rehabilitating, and special needs dogs, ToeGrips provide traction, confidence, and proprioceptive stimulus. Dogs with proprioceptive deficits will likely pull off ToeGrips; however, ToeGrips may be affixed to the nails with super glue as needed. ToeGrips are not intended for young, active dogs.

Sizing

Measuring the nails is an essential, one-time process for each patient. Improperly sized, ToeGrips may quickly fall off or migrate into the nail bed. For best results, nails should be appropriately trimmed before sizing and applying ToeGrips.

Application

Immerse sixteen ToeGrips in isopropyl alcohol. Apply wet, using the alcohol as a lubricant. Illustrated directions are located inside each package. Please keep in mind that ToeGrips must create the GripZone to enable traction. Correct placement is critical for success.

Client Education

Rarely, ToeGrips may shift into the nail bed and cause harm. Please advise your clients to monitor their dogs' ToeGrips for proper positioning. ToeGrips will need to be replaced every 1-3 months, depending on the dog's size, environment, and gait. The more abnormal the dog's gait, the faster the ToeGrips will wear.